RETREATY

www.dendist.net
Endodontics kit

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Retreatment of Two Bioceramic Sealers Included Two Different Percentages of Calcium Silicate Using Two Endodontic File Systems: An In Vitro Study

Table 2 Mean and standard deviations of needed time to attain the apex.

Materials	Test	RT	PUR	Statistical analysis
CeraSeal (CRS)	Time (min, s)	2.33 ± 0.81 ^{a,A}	3.83 ± 1.30 ^{a,A}	a < b
AH Plus Bioceramic (AHB)		1.65 ± 0.55 ^b	1.48 ± 0.29 ^b	No
Statistical analysis "t-test"		A > B	A > B	

Abbreviations: PUR, Profaper Universal Retreatment; RT, Retreaty.
Note: Superscript small letters indicate statistically significant differences in rows and capital superscript letters indicate statistically significant differences in columns.

In addition, RT demonstrated less time needed to attain the apex and less remaining materials at the coronal and middle thirds. No influence of the retreatment instrument of the

Materials and Methods

Materials

Two premixed calcium silicate-based sealers, AHB (Dentsply Sirona) and CRS (Meta Biomed), were used in the present study. The chemical composition and the calcium silicate percentages of each sealer are provided in Table 1.

In addition, two endodontic retreatment systems, Retreaty (Dental Perfect, Shenzhen City, Guangdong Prov-

During the retreatment preparation, from 1.5 to 1.5 Hz and 375 rpm (Dental Perfect, Shenzhen City, Guangdong Prov-)

Downloaded from jeb.oxfordjournals.com/ by guest on 04/11/2025